

PANEL INSTALLATION GUIDELINES

Initial Installation

1. Match the location of the mesh to the inventory list
2. Ensure that mesh is hung with ribbon toward the outside, snaps on the inside (bed side)
3. Mesh should be hung with the first carrier on the end cap with stationary hook as so the curtain stays close to the wall
4. Start snaps at the far left side and snap on a full panel (left to right from snap side)
5. Panels overlap by 2 snaps, add additional panels from left to right as needed
 - a. U shaped track typically has 3 panels (standard 68 " size)
 - b. L shaped track typically has 2 panels (standard 68 " size)
6. InstaGap panel is added between two panels when there is a lift
7. Ensure that mesh stays up and is not washed

Daily Use

1. Soiled panels are removed at patient turnaround or as per the routine changeout schedule
2. Unsnap panels from mesh (mesh stays on hooks)
3. Snap on new panels as per installation instructions above (panels are all the same size)
4. Soiled panels are to be sent to launder

**Panels should have a lifetime usage of approximately seven years.*

